Hillrom™

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**Hillrom™ Progressive Mobility® Program**

Initial Patient Assessment	Level 1: BREATHE	Level 2: TILT	Level 3: SIT	Level 4: STAND	Level 5: MOVE	
<p>Assess patient's mobility levels within 8 hours of admission to the ICU and reassess at least every shift.</p> <p>Start at level 1 if the patient meets any of these criteria or skip to level 2.</p> <p>PaO<sub>2</sub>/FiO<sub>2</sub> &lt;250</p> <p>Positive End-Expiratory Pressure (PEEP) ≥10 cm H<sub>2</sub>O</p> <p>O<sub>2</sub> Saturation &lt;90%</p> <p>Respiratory Rate (RR) Not within 10-30 per minute</p> <p>Cardiac Arrhythmias or Ischemia New Onset</p> <p>Heart Rate (HR) &lt;60 or &gt;120 beats per minute</p> <p>Mean Arterial Pressure (MAP) &lt;55 or &gt;140 mm Hg</p> <p>Systolic Blood Pressure (SBP) &lt;90 or &gt;180 mm Hg</p> <p>Vasopressor Infusion New or increasing</p> <p>Richmond Agitation Sedation Scale (RASS) +3</p> <p>Riker Sedation-Agitation Scale (SAS) &lt;-3</p>	<p><b>Level 1: BREATHE</b></p> <p>Patient Assessment: RASS -5 to -3; SAS 1-2 <i>(eg. cannot participate)</i></p> <p>Activities</p> <ul style="list-style-type: none"> <li>Maintain HOB ≥30°</li> <li>q2hr turning</li> <li>Consider continuous lateral rotation therapy (CLRT)</li> <li>Passive range of motion (ROM) 2 times/day</li> <li>Up to 20° Reverse Trendelenburg/Tilt Table with lower extremity exercises/retracting footboard Min 15 mins/Max 60 mins   1 time/day</li> </ul> <p>Move to Level 2 when the Patient...</p> <ul style="list-style-type: none"> <li>Has acceptable oxygenation/hemodynamics</li> <li>Tolerates q2hr turning</li> <li>Tolerates HOB &gt;30° or up to 20° Reverse Trendelenburg</li> </ul>	<p><b>Level 2: TILT</b></p> <p>Patient Assessment: RASS -3; SAS &gt;3 <i>(eg. opens eyes, may have profound weakness)</i></p> <p>Activities</p> <ul style="list-style-type: none"> <li>Maintain HOB ≥30°</li> <li>q2hr turning</li> <li>Passive/active ROM   3 times/day</li> <li>Up to 20° Reverse Trendelenburg/Tilt Table with lower extremity exercises/retracting footboard Min 15 mins/Max 60 mins   3 times/day</li> <li>Legs dependent 15-20 mins   3 times/day</li> <li>Physical therapist (PT) consultation 1 time/day</li> </ul> <p>Move to Level 3 when the Patient...</p> <ul style="list-style-type: none"> <li>Tolerates active-assistance exercises 2 times/day</li> <li>Tolerates lower extremity exercises against footboard/Up to 20° Reverse Trendelenburg</li> <li>Tolerates legs dependent / HOB 45°</li> </ul>	<p><b>Level 3: SIT</b></p> <p>Patient Assessment: RASS &gt;-1; SAS &gt;3 <i>(eg. weak but may move arms/legs independently)</i></p> <p>Activities</p> <ul style="list-style-type: none"> <li>Maintain HOB ≥30°</li> <li>q2hr turning (self/assisted)</li> <li>Active ROM   3 times/day</li> <li>Encourage activities of daily living As tolerated</li> <li>Full chair position (footboard on) 60 mins   3 times/day</li> <li>Dangling, if patient can move arm against gravity   As tolerated</li> <li>PT Occupational Therapy (OT) actively involved   1 time/day</li> </ul> <p>Move to Level 4 when the Patient...</p> <ul style="list-style-type: none"> <li>Tolerates increasing active exercise in bed</li> <li>Actively assists with q2hr turning or turns independently</li> <li>Tolerates full chair position 3 times/day</li> </ul>	<p><b>Level 4: STAND</b></p> <p>Patient Assessment: RASS &gt;0; SAS &gt;4 <i>(eg. weak but may tolerate increased activity)</i></p> <p>Activities</p> <ul style="list-style-type: none"> <li>Maintain HOB ≥30°</li> <li>q2hr turning (self/assisted)</li> <li>Active ROM   3 times/day</li> <li>Encourage activities of daily living As tolerated</li> <li>Full chair position (footboard off/feet on the floor)   3 times/day</li> <li>Stand attempts, if patient can move leg against gravity (use a sit-to-stand lift) 3 times/day</li> <li>Pivot to chair, if tolerates partial weight bearing   2 times/day</li> <li>PT/OT actively involved   1 time/day</li> </ul> <p>Move to Level 5 when the Patient...</p> <ul style="list-style-type: none"> <li>Can successfully comply with all activities</li> <li>Tolerates trial periods of full chair position (footboard off/feet on the floor) 3 times/day</li> <li>Tolerates partial weight-bearing stand and pivots to chair</li> </ul>	<p><b>Level 5: MOVE</b></p> <p>Patient Assessment: RASS &gt;0; SAS &gt;4 <i>(eg. weak but may tolerate increased activity)</i></p> <p>Activities</p> <ul style="list-style-type: none"> <li>Maintain HOB ≥30°</li> <li>q2hr turning (self/assisted)</li> <li>Active ROM   3 times/day</li> <li>Encourage activities of daily living As tolerated</li> <li>Patient stands/bears weight &gt;1 min 3 times/day</li> <li>Patient marches in place   3 times/day</li> <li>Ambulate to bedside chair to achieve "out-of-bed" (use a patient lift) 3 times/day</li> <li>PT/OT actively involved   1 time/day</li> </ul> <p>Continue to ambulate progressively longer distances as tolerated until patient consistently participates and moves independently.</p>	
	Assessed to Level 1 or 2	Progress to Level 2	Progress to Level 3	Progress to Level 4	Progress to Level 5	End Protocol

© 2021 Hillrom Services, Inc. ALL RIGHTS RESERVED. 3045-1415 18 AUG 2020 ENG - US

This tool is provided for education and discussion only. Each facility is responsible for the development, adoption and implementation of its own protocols. Follow protocols and rules adopted by your facility.

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[Kate E. Klein](#), [James F. Bena](#), [Nancy M. Albert](#), "Impact of Early Mobilization on Mechanical Ventilation and Cost in Neurological ICU" - American Journal of Respiratory and Critical Care Medicine 2015;191:A2293

Klein K, Mulkey M, Bena JF, Albert NM. Clinical and psychological effects of early mobilization in patients treated in a neurologic ICU: a comparative study. Crit Care Med. 2015 Apr;43(4):865-73. doi: 10.1097/CCM.0000000000000787. PMID: 25517476.

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© 2021 Hillrom Services, Inc. TOUS DROITS RÉSERVÉS. APR245101-FR-EMEA-R1 23-june-2021 French - EMEA